


RHINEHART DEVELOPMENT CORP

DAIRY RESEARCH PRODUCTS, INC.
Spencerville, Indiana 46788

Paddle position with handle toward you. Record individual quarters.

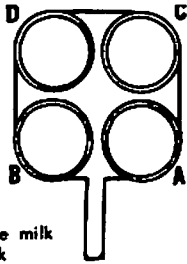

Acid/Alkaline Code
+ Purple color/alkaline milk
Y Yellow color/acid milk

mastitis code symbols

- Negative: Mixture remains fluid; no slime or sediment.
- T Trace: Mixture becomes slimy.
- 1 Weak: Mixture forms distinct sediment.
- 2 Distinct: Mixture thickens immediately; tends to form jelly.
- 3 Strong: Mixture forms jelly peak in center, even after swirling stops.

CALIFORNIA MASTITIS TEST

records on herd of

.....
(name)

COW Identification										
TEST #1 date:										
TEST #2 date:										
TEST #3 date:										
TEST #4 date:										
TEST #5 date:										
TEST #6 date:										